



NUMBRELLA

Your Pain - Our Shield

For :
WLM-NNPD-WBP-2024

Instruction Card

Doc No.
WLM-NNPD-WBP-2024-RevA

What is Numbrella?

Numbrella is a special device that utilizes an innovative two-point discrimination method to transform the experience of medical procedures involving injections or blood sample collections. This method leverages the body's sensory system to create a gentle and distracting touch that effectively minimizes the sensation of pain and discomfort during injections or Needle pricks.

Two-Point Discrimination: The two-point discrimination method involves applying two separate points of touch to the skin simultaneously, activating the body's sensory receptors and altering the perception of pain. By strategically positioning these touch points around the injection site or blood collection area, Numbrella effectively distracts the brain from processing the sensation of the needle, resulting in a reduced perception of pain.

Gentle and Distracting Touch: Numbrella's touch is gentle yet purposeful, designed to create a comforting sensation that overrides the discomfort associated with medical procedures involving needles. The device emits a subtle vibration or pressure around the injection site, creating a soothing and distracting sensation that helps users focus their attention away from the needle.


Redirected Focus: As users experience the gentle touch of Numbrella, their focus is redirected away from the sensation of pain, anxiety, or fear typically associated with medical procedures involving needles. Instead of fixating on the needle, users may perceive a gentle numbing or tingling sensation, making the entire process more tolerable and reassuring.

Comforting Shield: Numbrella acts as a comforting shield for individuals undergoing medical procedures, enveloping them in a sense of tranquility and reassurance. The device creates a supportive environment that promotes relaxation and confidence, allowing users to approach medical procedures involving needles with greater ease and peace of mind.

Universal Application: Numbrella's two-point discrimination method is universally applicable to a wide range of medical procedures, including injections, vaccinations, and blood sample collections.

Whether used in clinical settings, at home, or during community health events, Numbrella provides consistent comfort and support for individuals of all ages.

Experience the Transformative Power of Numbrella Today!

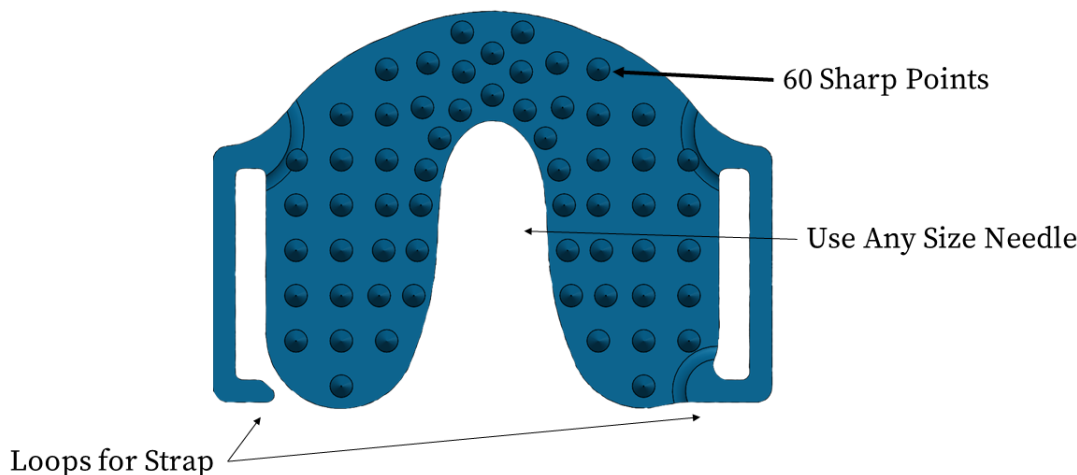


Features and Advantages

- Numerous Gentle Touch Points Surrounding the Injection Site, Ensuring a Comfortable Experience.
- Non-invasive and Pain-Free Application, Making Medical Procedures More Tolerable.
- Suitable for All Ages, Offering Relief to Patients of Any Generation.
- Adaptable Design: Can be Utilized at Home, Clinics, Diagnostic Centers, Vaccination Clinics, or Any Setting Requiring Needle-Based Procedures.
- Effortless Operation, Making it Easy for Anyone to Use.
- Streamlined, Swift, and Highly Effective, Providing Instant Relief During Medical Procedures.
- Portable and Lightweight: Compact design allows for easy transport, ensuring comfort wherever you go.
- Versatile Compatibility: Works seamlessly with various injection types and needle sizes, accommodating diverse medical needs.


Indications for Use

Numbrella's fundamental purpose is to alleviate anxiety and fear associated with needles by providing distraction and comfort to patients. Its versatile applications extend across various medical settings, including:


- Drawing Blood Samples
- Giving Intramuscular Injections
- Giving Subcutaneous Injections
- Giving Vaccinations
- Giving Intradermal Injections.
- Cosmetic Procedures
- Chronic Disease Management, like Insulin for Diabetes
- Emergency Medical Services

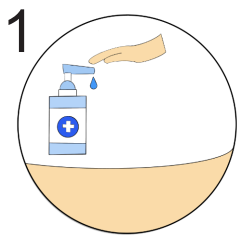

Ensure thorough cleansing of the injection site to uphold sterility.

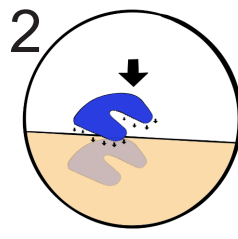


Position Numbrella over the injection site, ensuring its pointed ends face the skin.

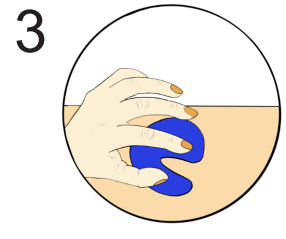


Secure Numbrella firmly onto the skin's surface by applying pressure or using the strap.


DO NOT PRESS HARD

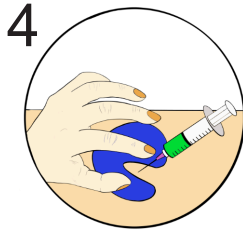

Administer the injection or draw the sample as required, then carefully remove the needle.

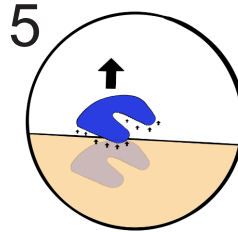

Lift Numbrella delicately from the skin, ensuring it is removed only after the needle.

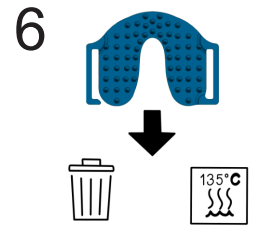

Dispose of Numbrella responsibly or sterilize it for reuse, maintaining proper hygiene standards.

Tips

- Educate Patients:** Provide clear and concise instructions to patients about the purpose and use of Numbrella. Explain how it works and the benefits it offers in minimizing pain and anxiety during needle-related procedures.
- Positioning:** Position Numbrella near the injection site or blood draw location to ensure optimal distraction and comfort for the patient. Place it within the patient's line of sight to draw their attention away from the needle.
- Engage Patients:** Encourage patients to interact with Numbrella by touching or holding it during the procedure. This active engagement can help enhance the distraction effect and promote a sense of control and comfort.
- Monitor Patient Response:** Pay attention to the patient's response while using Numbrella. Observe their level of relaxation and comfort during the procedure and adjust the placement or usage of Numbrella as needed to optimize its effectiveness.
- Use Positive Reinforcement:** Provide positive reinforcement and encouragement to patients who use Numbrella during procedures. Acknowledge their efforts to manage anxiety and discomfort and praise them for their cooperation.
- Ensure Proper Hygiene:** Numbrella is a Single Use Device. However, incase, if it is used again, follow standard hygiene protocols when using Numbrella, including cleaning and disinfecting the device between uses to prevent cross-contamination and ensure patient safety. Ideally Numbrella should be used as a single used device. We strongly advocate to use Numbrella as a Single use device.
- Offer Support:** Be attentive and supportive to patients who may still experience discomfort or anxiety despite using Numbrella. Offer reassurance and comfort throughout the procedure to help alleviate their concerns.
- Seek Patient Feedback:** After the procedure, ask patients for feedback on their experience using Numbrella. Inquire about their level of comfort and satisfaction to identify any areas for improvement or adjustment in its usage.
- Continued Education:** Stay updated on best practices and new developments related to using Numbrella or similar distraction devices in clinical settings. Attend training sessions or workshops to enhance your skills in effectively incorporating these tools into patient care.

Warning: Numbrella is intended for distraction and comfort purposes only during medical procedures involving needle insertion. It is not a substitute for proper medical care or supervision. Users should always follow healthcare provider instructions and guidelines when using Numbrella. Consult a healthcare professional before using Numbrella if you have any pre-existing medical conditions or concerns. Keep Numbrella out of reach of children and pets to prevent accidental ingestion or misuse. Do not reuse Numbrella, as it may increase the risk of disease transmission. If reused, and there is a spread of disease, we are not responsible for it. Numbrella has sharp points, and these can puncture or injure the patient, at most care must be taken when using this device. Do not use Numbrella if the device appears damaged or malfunctioning. Use Numbrella only as directed and discontinue use if irritation or discomfort occurs. By using Numbrella, you acknowledge and accept the inherent risks associated with its usage and agree to hold harmless the manufacturer, distributors, and retailers from any liability arising from the use of this product. Numbrella is not intended to diagnose, treat, cure or prevent any disease. Numbrella is not sold as a medical device. Effectiveness of Numbrella varies from person to person.

Manufactured By:

White Lotus Merchandise LLP.

Website: www.whitelm.com

Address: F202 6-3-663/7 SaaiPriya Apartments,
Zaffaralibhag, Somajiguda, Hyderabad,
Telangana, India 500082

